

General Advice: These instructions are general advice only and do not take into account your specific circumstance. These instructions are to be used in conjunction with the information provided by your health professional and any instructions provided by the equipment manufacturer. Do not exceed the Maximum User Weight specified by the manufacturer. If you are unsure on how to use or fit a piece of equipment, please contact your Health Professional or Country Care Store.

Swivel Cushion Use

- A swivel cushion is used to assist individuals in getting in and out of a car, chair, or other seating surface. It typically consists of a round base that can rotate 360 degrees, and a seat cushion that is attached to the base
- The swivel cushion is designed to reduce the need for twisting or turning when getting in and out of a seated position. Some swivel cushions also have a built-in handle or other feature that can be used for added support and stability when getting in and out of the seat
- Swivel cushions can be used in a variety of settings, including in the home, car, office, or other public spaces



Installation

- Place swivel cushion in desired position on car seat or a flat stable position if required for use elsewhere. If not using in the car ensure swivel seat does not slip on the surface it is sat on

How to Use a Swivel Cushion

1. Sit on swivel cushion, feet flat on the floor and swivel on cushion to enter and exit the seat
- **Note:** It's important to seek guidance and advice from your Healthcare Professional or a member of the Country Care Group service team before use to ensure correct fit and usage of all assistive technology equipment

MAINTENANCE & CLEANING

- Please check the product label / user manual for specific cleaning instructions
- If the equipment appears damaged, **DO NOT USE**. It may require replacing